


Transflux™


The Standard in Contrast Administration

ONCE A DAY

1 Insert Syringes

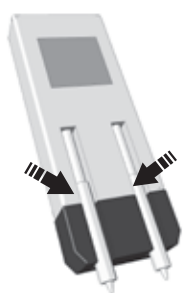


2 Connect Transset to injector

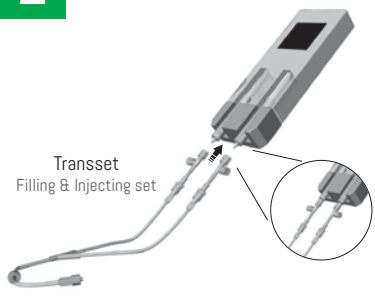

Transset Filling & Injecting set


3 Connect system to contrast medium and saline reservoirs + flush with saline

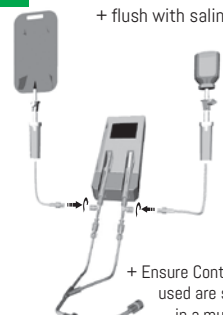

+ Ensure Contrast and Saline used are suitable for use in a multi-use system!


ONCE PER PATIENT

1 Hold Transflux™ upright and flush




push to flush

2 Insert needle into vein or via port-a-cath




PATIENT

3 Connect Transflux™ to needle




4 Check 'Blood Reflux' using disconnect feature

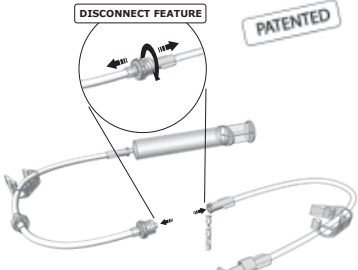

DISCONNECT FEATURE PATENTED


PATIENT

5 Reconnect and check all connections

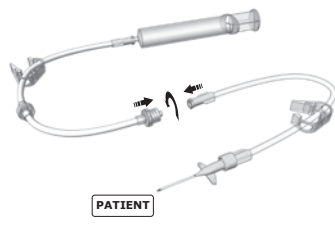

PATIENT


6 Connect Transflux™ to Transset™ and injector



PATIENT

➡ System ready for injection!

AFTER INJECTION

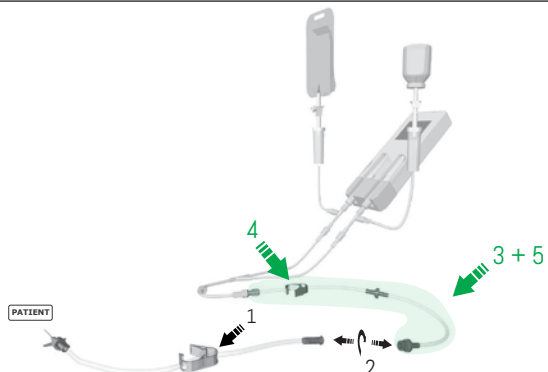

1 ➡ close the clamp

2 ➡ disconnect valve

3 ➡ leave proximal part of the Transflux™ connected to the Transset™ as a safety cap.

4 ➡ close after use to indicate that Transflux™ has been used

5 ➡ remove this part before installing the new Transflux™ for a new patient